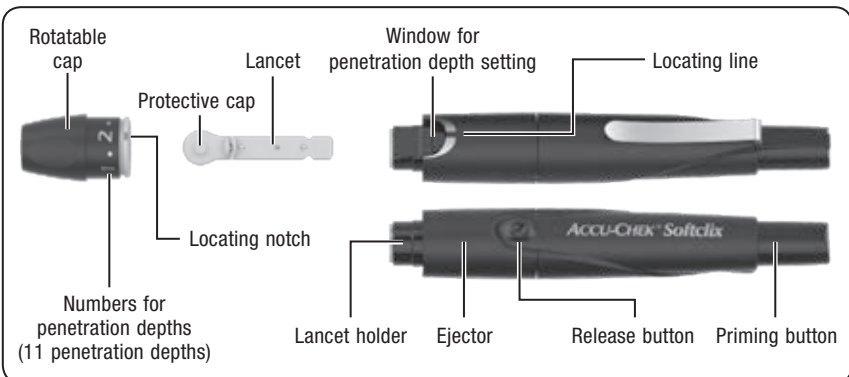
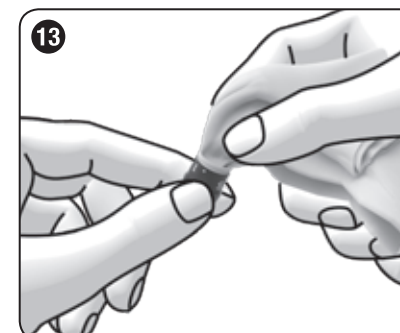
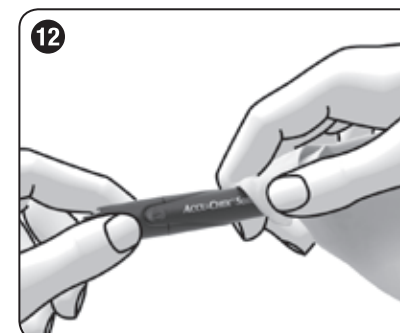
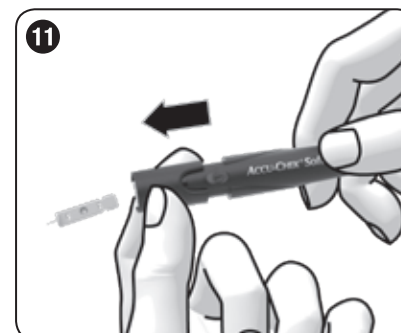
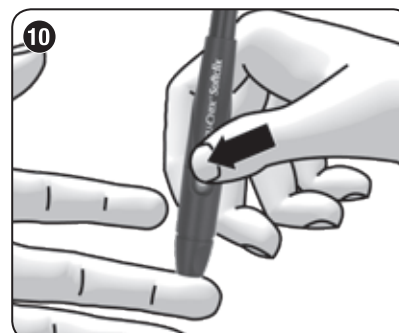
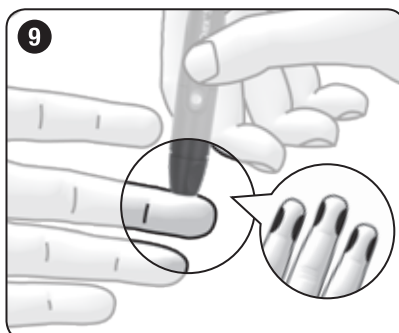
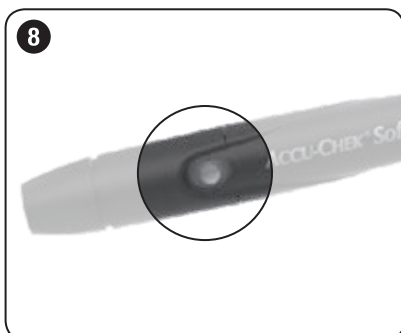
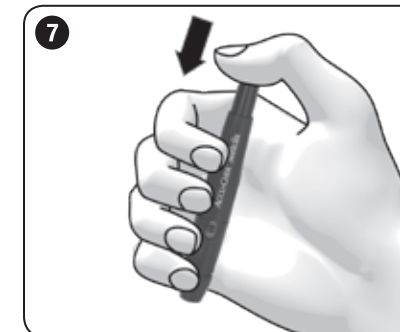
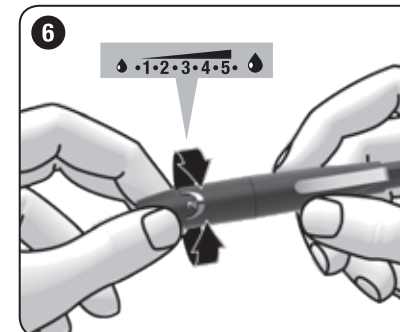
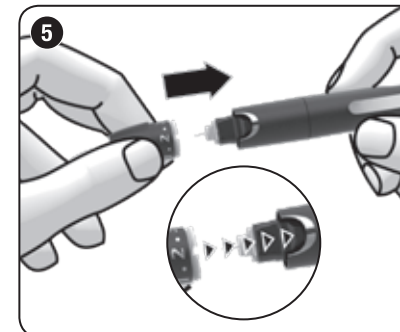
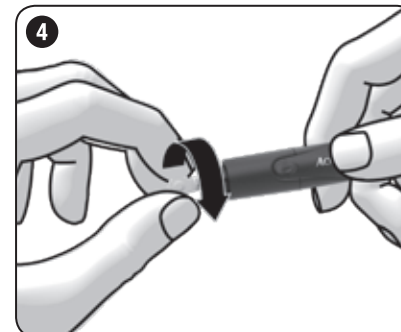
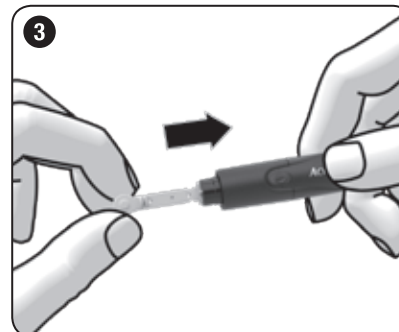
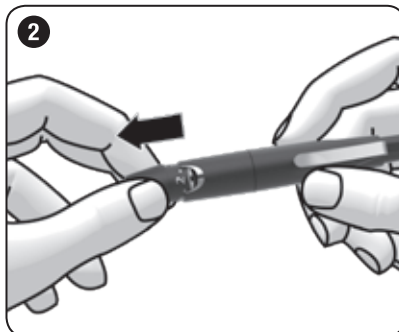
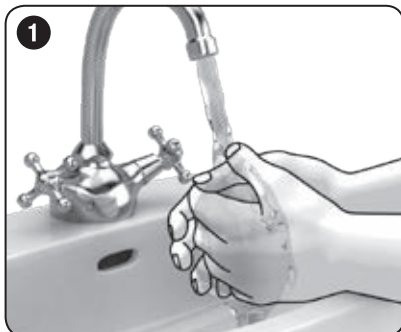
ACCU-CHEK® Softclix

Lancing device

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EN Instructions for use

The Accu-Chek Softclix lancing device is a device for simple, virtually pain-free collection of capillary blood from the fingertips. The lancing device has 11 different penetration depths so that you can adjust the penetration depth to the texture of your skin.

These instructions for use feature the following 2 symbols:

This symbol indicates a **possible risk of injury or of damage to your health.**

This symbol draws attention to actions that could result in **damage to the lancing device.**

• The **Accu-Chek Softclix lancing device is intended for personal use only!**

It is only allowed to be used to obtain blood from **one person.**

There is a risk of infections being transmitted if the lancing device is used by other people, even by family members, or if healthcare professionals use this lancing device to obtain blood from different people. This lancing device is therefore not suitable for professional use in healthcare facilities and institutions.

• Keep the lancing device with all its components out of reach of children under 3 years. There is a risk of suffocation if small parts (e.g. caps or similar objects) are swallowed.

• If the lancing device falls with a lancet inserted, the lancet may become loose in the lancet holder. You cannot use it anymore to obtain blood. In rare cases, the lancet could protrude from the opening in the cap and you could injure yourself on it. Therefore, do not touch the front of the cap. If the lancing device falls, pick it up with care. Remove the cap from the lancing device. Pay attention to the lancet so that you do not injure yourself on it. Always eject the lancet and discard it.

Only use Accu-Chek Softclix lancets in the Accu-Chek Softclix lancing device. Other lancets may impair the operation of the lancing device or damage the lancing device permanently.

Preparation

► Wash your hands with warm water and soap before you obtain blood (see **1**). Dry your hands thoroughly. This reduces contamination of the puncture site and stimulates blood flow.

Inserting the lancet

You have to insert a lancet to be able to obtain blood with the lancing device.

► Remove the cap from the lancing device (see **2**).

► Insert a new lancet into the lancet holder as far as it will go (see **3**). The lancet must audibly click into place.

► Twist the protective cap off of the lancet (see **4**).

► Place the cap back on again. The locating notch in the cap must be directly aligned with the locating line on the lancing device (see **5**). The cap must audibly click into place.

Setting the penetration depth

You can set 11 penetration depths (0.5 to 5.5). The penetration depth set is shown in the window. The larger the number, the greater the penetration depth.

If you have no experience with this lancing device, we recommend setting a low penetration depth, such as penetration depth 2.

► Rotate the cap until the desired penetration depth is set (see **6**).

Priming the lancing device

In order to be able to obtain blood, you must prime the lancing device.

► Press the priming button down as far as it will go (see **7**). Do not press the release button simultaneously.

The lancing device is primed when the centre of the release button is yellow (see **8**).

Only prime the lancing device just before obtaining blood.

Obtaining a blood drop

In general, blood can be obtained from any finger. Certain fingers can be unsuitable if, for example, a skin or fingernail infection is present. We recommend obtaining capillary blood from the sides of the fingertips as these areas are the least sensitive to pain.

• Always use a new lancet when you obtain blood. In doing this, you reduce the risk of infection, and obtaining blood remains virtually pain free.

• **Only** use the lancing device with the cap on. Without the cap, the lancet penetrates too deeply and the puncture can become painful.

► Press the lancing device firmly against the selected puncture site (see **9**).

► Press the release button (see **10**).

The lancet is released and it penetrates the skin.

► Massage the finger in the direction of the fingertip to encourage a blood drop to form.

The amount of blood that emerges depends on the penetration depth and the pressure used to hold the lancing device against the skin.

If not enough blood emerges, apply more pressure on the lancing device when you next obtain blood. If that is not sufficient, increase the penetration depth progressively in addition. If too much blood emerges, decrease the penetration depth.

► Test your blood glucose immediately after you obtained blood, according to the instructions for use for your blood glucose meter.

We recommend protecting the puncture site from contamination afterwards.

Ejecting the used lancet

► Remove the cap from the lancing device (see **2**).

► Slide the ejector forward (see **11**). The used lancet is ejected.

► Place the cap back on again.

► Dispose of used lancets according to local regulations.

Dispose of used lancets in such a way as to prevent injury from the needles.

Storing the lancing device

Do not store the lancing device in a primed state or at extreme temperatures, for example, in a hot car. This can impair the operation of the lancing device.

Store the lancing device only without a lancet inserted.

Cleaning and disinfecting the lancing device and cap

To prevent infections from being transmitted, you must clean and disinfect the lancing device and cap regularly:

• At least once a week

• When there is blood on the lancing device or cap

• Always before someone else handles the lancing device, for example, to assist you.

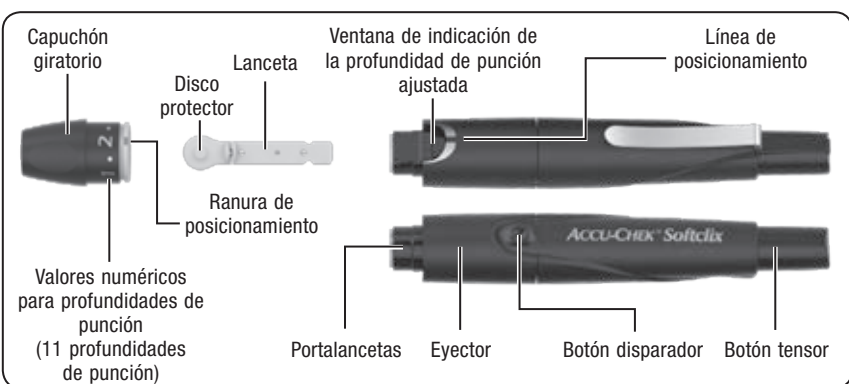
You need lint-free cloths in which you can wrap the whole lancing device, water and 70 % isopropanol.

Before you disinfect the lancing device and the cap, you first have to clean them to remove blood and other dirt.

To avoid damaging the lancing device and the cap:

• Do not dip the lancing device and the cap in the fluids.

• Use only water for cleaning and only 70 % isopropanol for disinfecting.

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Last update

2019-04

Customer Support and Service Centre

United Kingdom

Roche Diabetes Care Limited
Charles Avenue, Burgess Hill
West Sussex, RH15 9RY, **United Kingdom**
Accu-Chek Customer Careline 1)
UK Freephone number: 0800 701 000
ROI Freephone number: 1 800 709 600

1) calls may be recorded for training purposes
Some mobile operators may charge for calls to these numbers.

www.accu-chek.co.uk

www.accu-chek.ie

Roche Diabetes Care Australia Pty. Ltd.

24-32 Lexington Drive, Bella Vista
NSW 2153, **Australia**
ABN 69 602 140 278
Accu-Chek Enquiry Line: 1800 251 816
www.accu-chek.com.au

Canada

Accu-Chek Customer Care: 1-800-363-7949
www.accu-chek.ca

Caution, refer to safety-related notes in the instructions for use accompanying this product.

Use only once

Use by

Sterilized using irradiation

Manufacturer

Catalogue number

Batch code

Global Trade Item Number

This product fulfils the requirements of the European Directive 93/42/EEC on medical devices.

CE 0123

ACCU-CHEK and SOFTCLIX are trademarks of Roche.

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© 2019 Roche Diabetes Care